


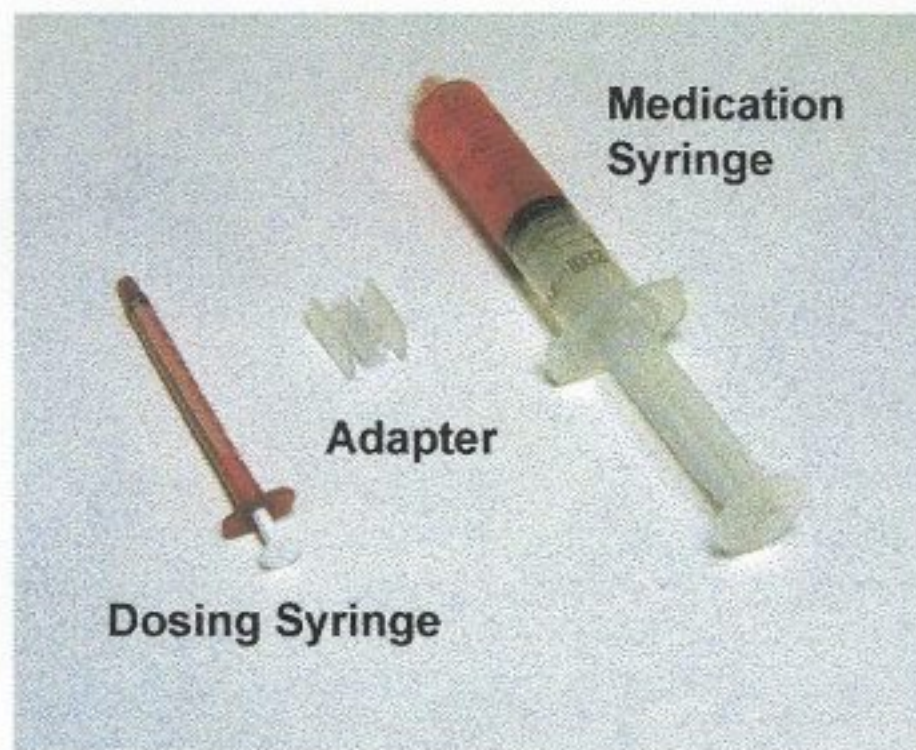


## **INSTRUCTIONS FOR USING THE DOSING SYRINGE AND ADAPTER**

1. Shake Medication Syringe well for 10-15 seconds. Remove black tip from Medication Syringe.



2. Screw the Adapter's small locking end with tabs into the Medication Syringe's tip until tight.

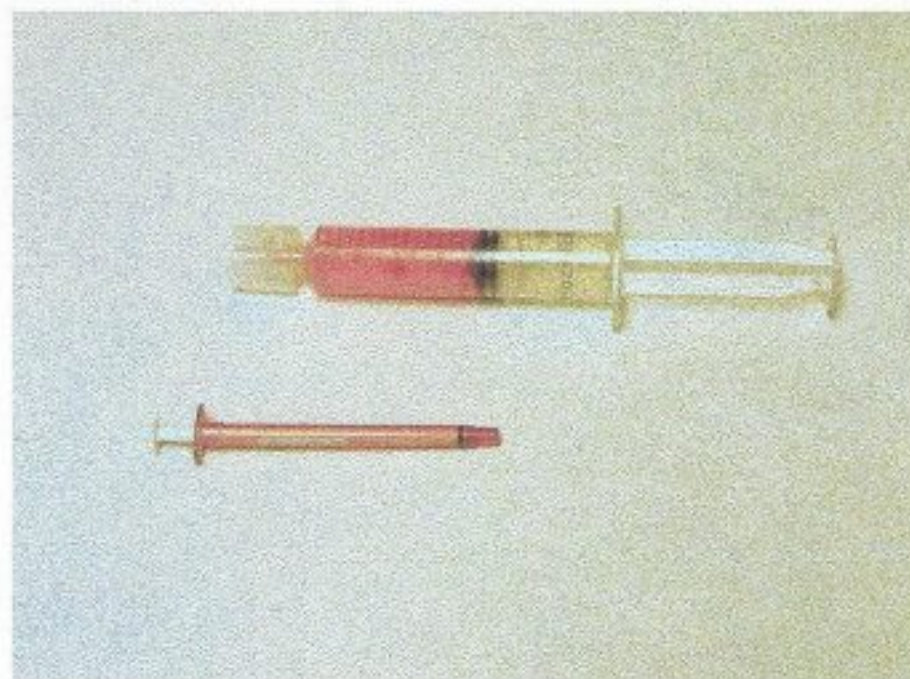

3. Push the Dosing Syringe into the round end of the Adapter until tight. Push on the plunger of the Medication Syringe while gently pulling on the Dosing Syringe's plunger to transfer medication from Medication Syringe through adapter into Dosing Syringe.



4. Remove the Dosing Syringe and administer medication to patient according to prescription label. Remove the Adapter and replace the black tip on the Medication Syringe for storage.

