


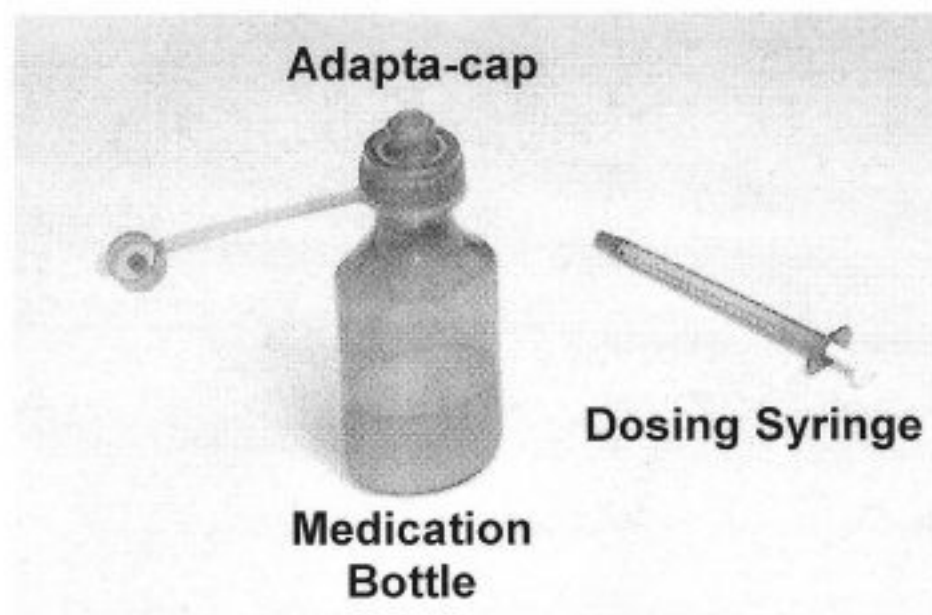


INSTRUCTIONS FOR USING THE DOSING SYRINGE AND ADAPTA-CAP


1. Shake Medication Bottle well for 10-15 seconds. Remove the white child-resistant cap from the Medication Bottle. Screw on the Blue Adapta-cap until tight.



2. Insert the Dosing Syringe into the Adapta-cap until tight.

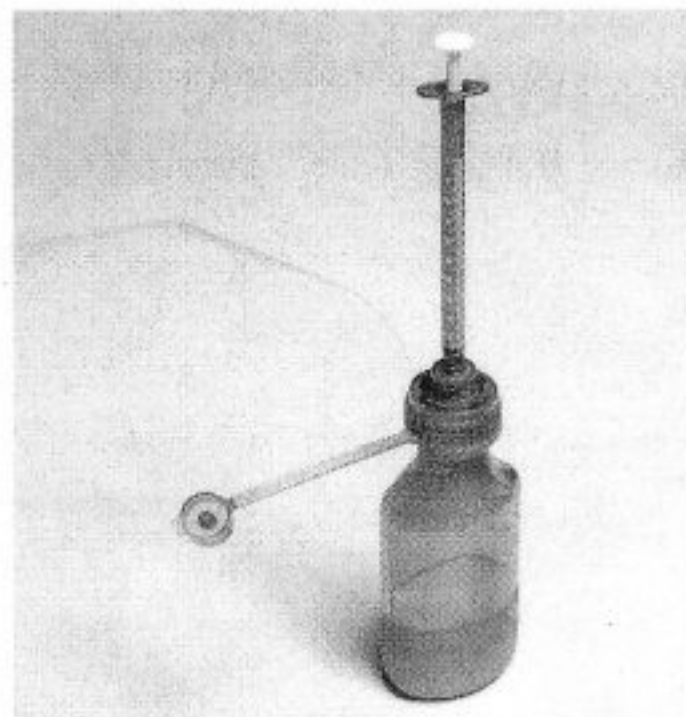

3. Turn the bottle with syringe upside down. Draw out the medication into the Dosing Syringe.

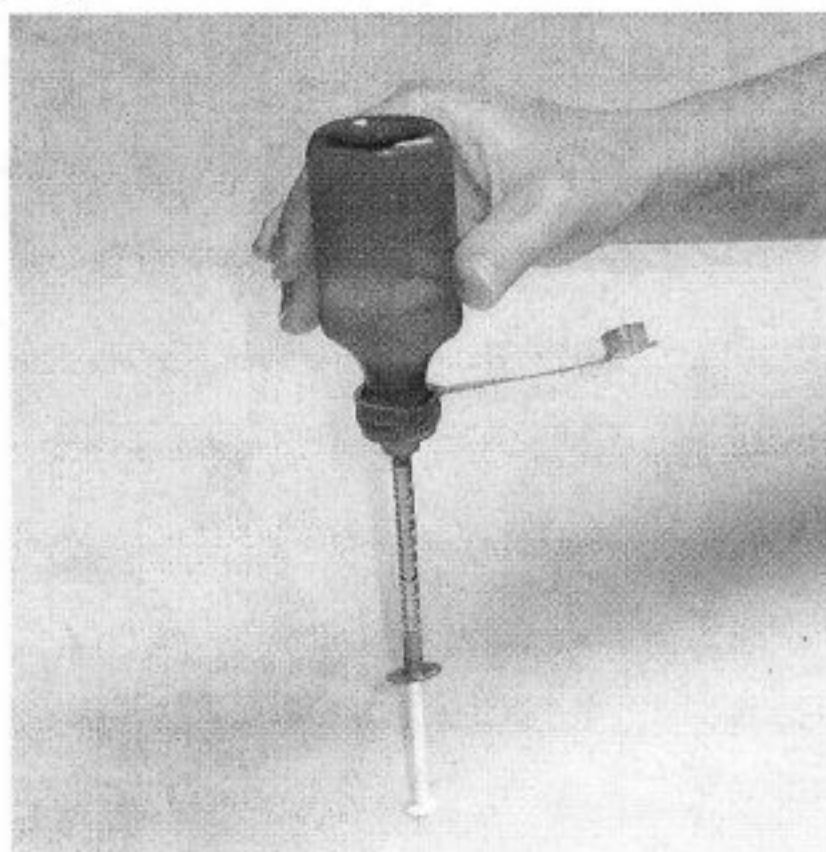

4. Place the bottle upright and remove the Dosing Syringe. Administer Medication to patient according to prescription label.

Replace Blue Adapta-cap with white child-resistant cap for storage

